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D - TP + TPL

Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Instructies

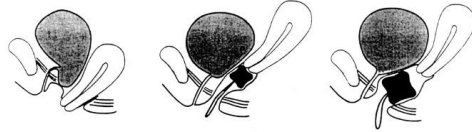
Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Tandem Pessary

Application: The tandem pessary is made of two cubes of flexible silicone coupled to each other. **Patients with grade 3-4 uterine or vaginal vault prolapse may be treated with the tandem pessary.** The reduction of the prolapse may not be possible if the vagina cannot retain a single cube pessary. Similar to the cube pessary, the vacuum effect of the concave surface achieves a good adhesion. In addition, the tandem pessary can be useful in case of stress incontinence whereby the lower cube takes on the supporting function of the urethra and the upper cube mainly assures the adhesion of the pessary.

Sizes: The edge lengths of the cubes determine the size of the tandem pessary. Since 6 different sizes of cubes are available, ranging from a minimum of 25mm edge length (size 0) up to maximum of 45mm (size 5), there are various possibilities of combinations. The cubes should be large enough to remain in their position, also while pressing, coughing and moving; on the other hand, edge lengths which are too large may press on the bladder or intestine. Under these conditions, it will increase difficulties to remove the pessary. Treatment should be started with smaller cube sizes, a correction may be necessary after a few days or weeks. Each combination can be chosen. In the case of an isolated uterine prolapse, smaller cubes will be combined and will be placed in the upper third of the vagina (picture in the middle). In cases of an additional prolapse of bladder and intestine larger cubes will be combined (right picture). A well fitted pessary should be experienced as symptomatic relief without any uncomfortable complains.

Use: In general, tandem pessaries are changed by the patient herself, after having been instructed by a physician or a nurse. **The pessary should be removed daily in the evening and inserted again in the morning.** Thus the vaginal tissue can recover over night. In some patients with a planned operation, it may make sense to leave the pessary over night to improve tissue circulation. In cases with a large prolapse, one can use a smaller pessary over night, or the pessary may only be removed approximately once a week by a nurse and re-inserted in the morning with estriol cream. Irrespective of the age of the patient, it is of advantage to spread the estriol cream on 2 or 3 edges of the cube before insertion. This will aid in the insertion, but also improve tissue circulation and recovery of a vagina flora. Exclusively creams containing estriol should be used. Since one needs a relatively large amount of cream (large surface) to reach a gliding effect, we recommend the use of a low dispensed cream of 0.1 mg estriol/1 g cream, in rare cases of 0.5 mg estriol / 1 g cream.

For insertion of the pessary the woman may put one foot on a stool or the edge of a bed, in a position comparable to the insertion of a tampon. If this is too difficult, it may be sufficient to insert the pessary while spreading the legs, if necessary while leaning against a wall or while lying and thereby to push the cube as far as possible up into the vagina.

For removing the pessary the woman has to pull it by its thread to varying directions, if necessary also while slightly pressing - until a resistance is felt by the pelvic floor muscles. With the forefinger and middle finger the patient may loosen the edges of the cube to remove the vacuum effect and pull the pessary further down.

Side Effects/Complications: If the thread should exceptionally tear, the pessary has to be removed by a physician. In rare cases with a large cystocele the cube pessary may manifest a stress incontinence, and is a warning sign that the same effect might be achieved by pelvic floor suspension surgery. In this case, the pessary should be turned with the edge into the region between bladder and urethra, or one should choose another model (urethra or urethra bowl pessary). Contraindications for estriol creams should be considered (e.g. pregnancy, breastfeeding, estrogen-dependent tumors).

If the patient is not able to change the pessary by herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar).

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